


1

Preparation


Check balloon

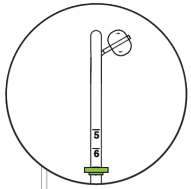

Keep syringe attached




Fill device using well in tray package

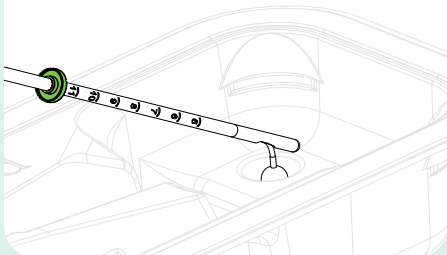


2

Place Device

Retract catheter
(green slider)

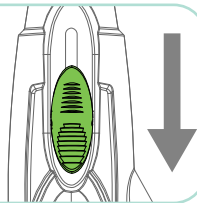


Insert to fundus

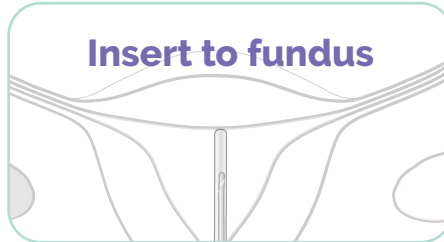

Advance catheter
(green slider)



Inflate balloon

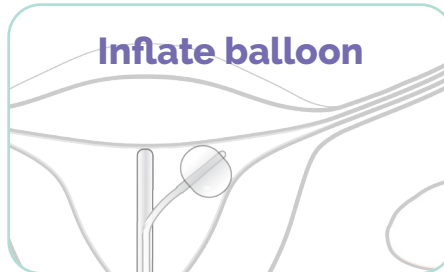

3

Deliver Sperm

Target delivery to selected fallopian tube



Optional
contralateral side


4

Remove Device

Deflate balloon

Retract catheter
(green slider)


Remove & discard

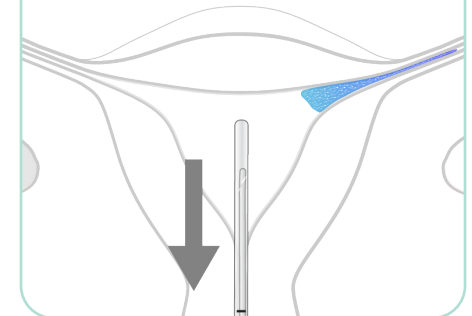

Scan QR code for
additional training



Product Features

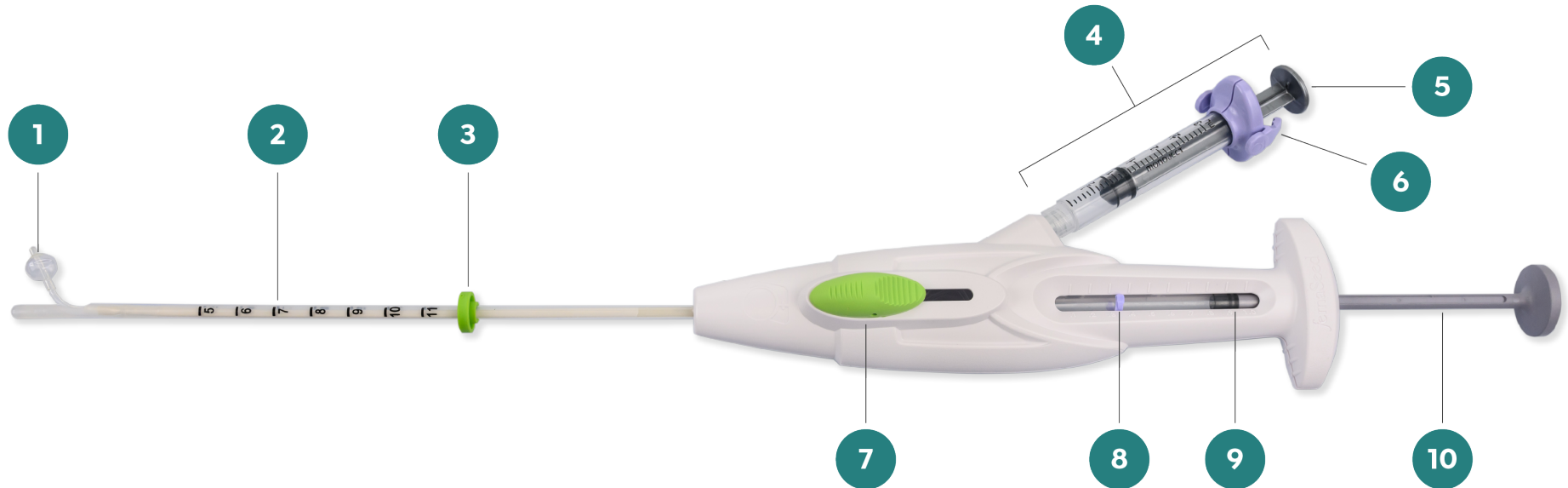

- 1. Curved transfer catheter:** Specimen fills from tip, balloon placed in uterine cornu (no catheterization)
- 2. Guide catheter:** Placed at uterine fundus for entire procedure
- 3. Flange (optional):** Used to indicate uterine depth after sounding
- 4. StopLock™ balloon inflation syringe**

- 5. Piston:** Used to deliver air for balloon inflation
- 6. Clips:** Used to deflate balloon
- 7. Slider:** Moves transfer catheter in and out of the guide catheter
- 8. Marker:** To note volume if delivering to contralateral tube
- 9. Built-in syringe:** Indicator for specimen filled into device
- 10. Plunger:** Used to deliver specimen